

mRidium™ Accessories


Infusion Bypass Set

When patients arrive in the MR requiring continued controlled IV therapy...

Transfer of IV pumps without disconnecting existing tubing is easy as **1-2-3.**

Quickly change over pre-existing non-MRI Infusion Pumps without having to tear down lines, break sterility, and increase cost.

The **mRidium™ 1055 MRI Infusion Bypass Set** answers the difficult problem of how to switch over hospitals' general non MRI approved infusion pumps to the new 3850 mRidium™ MRI Smart Infusion Pump without loss of existing tubing.

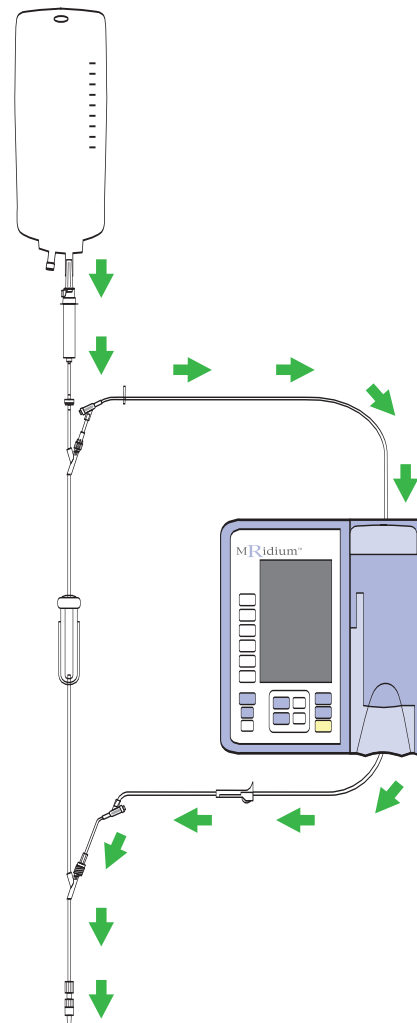


Features

- Minimal risk of sterility compromise
- Universal cost effective set design
- Built in set based free-flow protection
- Latex-Free Infusion Set
- Two Needle Free Valve Ports Included
- Allows for delivery of secondary fluids
- Sterile, Non-Toxic and Non-Pyrogenic
- Easy for clinicians to use
- MRI Safe
- DEHP-Free
- Low-Priming Volume


As easy as....

STEP 1

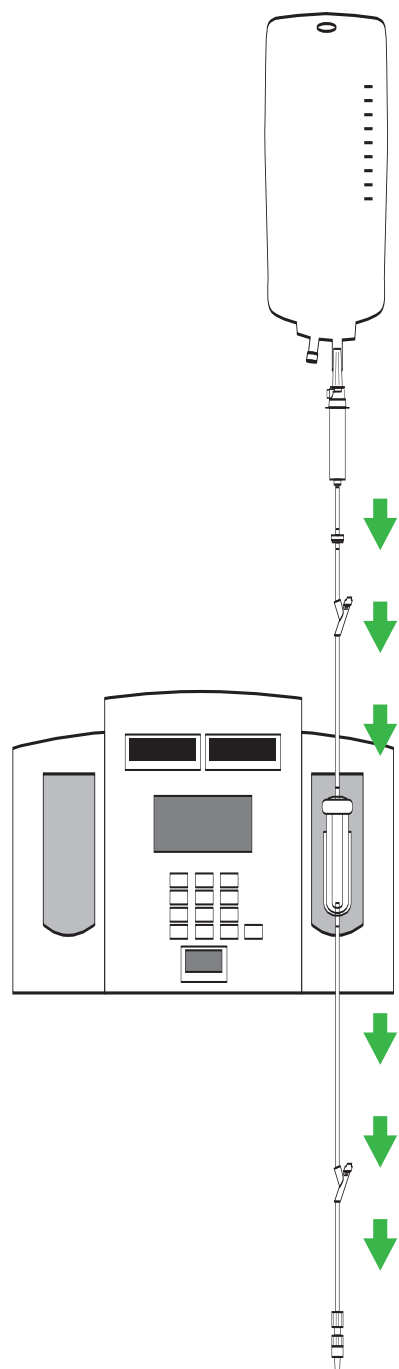


Patient arrives in MRI with non compatible IV Pump.

STEP 2

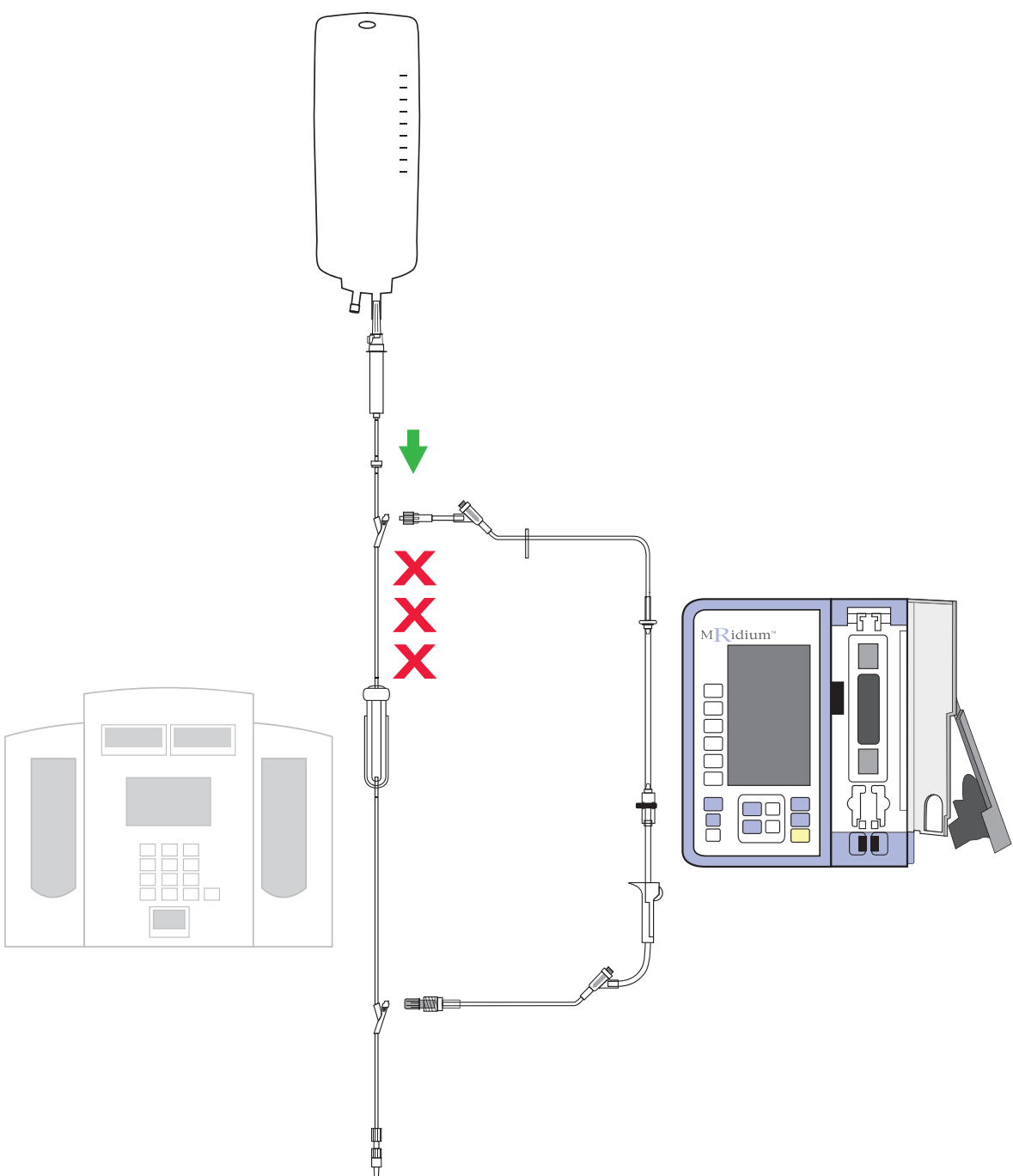

Attach **Bypass** set to the patient's current IV set, remove non MRI compatible pump, prime set, and insert into the mRidium™ MRI Infusion Pump.

STEP 3

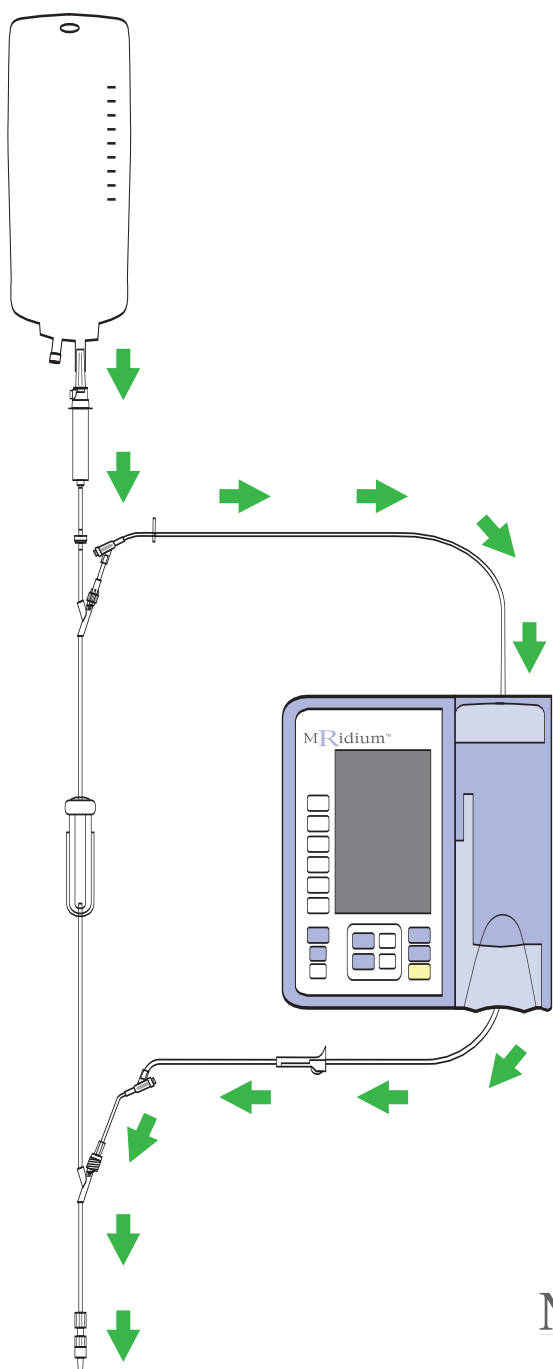

Program mRidium™ MRI Infusion Pump and start infusion.

Designed for MRI Safety

Using the **mRidium™ 1055 MRI Infusion Bypass Set** you can easily, safely, and cost-effectively switch from your magnetic non MRI approved infusion pumps to the new 3850 mRidium™ MRI Smart Infusion Pump without having to completely change out existing tubing.

It's a hassle to re-prime IV lines, not to mention the increase in cost by adding set after set to the bottom line. Each time you disconnect the IV line from the patient you increase the risk of infection.

The integrated needle-free valves can be activated only by a standard luer, eliminating the need for needles. This diminishes the accidental transmission of blood borne pathogens.

For more information, contact your IRadimed Sales Representative at 1-866-677-8022, or visit our web site at www.iradimed.com

mRidium™
1055 MRI Infusion Bypass Set

MRidium™

MRI Infusion System



Safe MRI
infusion
through
smart
technology

Deliver IV fluids
in MRI easily,
safely, and
cost-effectively.

MRidium™ 1057 Syringe Adapter
Infusion Set from Iradimed Corp.



MRidium™ 3855 Wireless
Remote Control



MRidium™ Infusion System Accessories

- 3855 Wireless IV Pump Remote Control
- 3851 Side Car, modular second pump channel
- 1055 ByPass Infusion Set
- 1056 Standard Infusion Set
- 1057 Syringe Adapter Infusion Set
- 1119 MR IV Pole
- 1133 Battery Pack

MAGMEDIX INC
160 Authority Drive
Fitchburg, MA 01420
866-646-3349
info@magmedix.com
www.magmedix.com