



TO CTDI + DOSIMETRY

TO CTDI+ permits the user to measure the dose index for computed tomography systems. The test object consists of three PMMA cylinders, one cylinder represents the paediatric head (100mm diameter) which, when nested into the next represents the adult head (160mm diameter) which, when nested inside the other, represents the body (320mm diameter). The length of each cylinder is 140mm. Each cylinder contains 13.0mm diameter holes and comes with two adaptor tubes to permit the use of either 8mm, 10mm or 13mm diameter radiation detectors, used to take the dose measurements. There is a hole at the centre of the cylinders and one at each 90° interval, centred 10mm from the edge of the cylinder. TO CTDI includes 13 PMMA pegs used to fill the unoccupied holes and a phantom support for each size of cylinder.

TO CTDI has been designed in accordance with standards IEC 61223-3-5, IEC 61223-3-6 and IEC 60601-2-44.



TO CTDI enables the following tests to be made:

- CTDI₁₀₀ (centre)
- CTDI₁₀₀ (peripheral)
- CTDI_w (weighted CTDI₁₀₀)
- CTDI_{vol} (weighted CTDI_w)
- CTDI_{free air}

TO CTDI nested head and body phantom is supplied in a trolley case with a telescopic handle.

For more information or to find your regional distributor please visit our website